
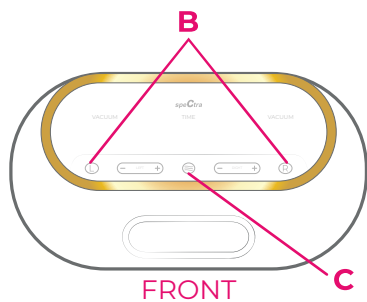


What's included with the Synergy Gold Portable?

- SG Portable Double Adjustable Electric Breast Pump
- 24mm Breast Flanges (2)
- 28mm Breast Flanges (2)
- Gold SpectraWide-Neck Storage Bottles with discs, caps, and covers (2)
- Duckbill Valves (2)
- Backflow Protectors (2)
- Tubing with Adapter Ends (2)
- 5 Volt AC Power Adapter (1)

Pump Function and Highlights

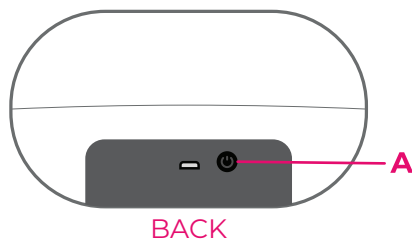

The **power** button (**A**) is located at the back of the pump.

Press the **L** and **R** buttons (**B**) to turn on independent motors for double pumping.

We recommend starting your pumping session on massage mode, indicated by the **3 wave** button (**C**) on the screen.

On massage mode, you will have 5 levels of vacuum that can be independently adjusted .

Cycle is set to quick speed for letdown.

Press the **3 wave** button to switch from massage to expression, indicated by the milk drop icon on the screen.

On expression mode, you will have 12 levels of vacuum and a preset cycle speed.

To stop double pumping, press the **power** button and then remove the flanges. To stop pumping on one side, select either the **L** or **R** button.

The touch-sensitive chime can be disable / enabled by holding down the **3 wave** button for 4 seconds.

The Spectra Synergy SG Portable maintains the same high standards as other Spectra Baby USA models with its trusted closed system, Natural Nursing Technology and 2-year warranty.

Unique features of the Spectra Synergy SG Portable



Rechargeable
portable with built in battery



Double Adjustable
individual vacuum settings for each breast



Massage Mode
simulates the natural suckling of baby



Closed System
prevents backflow into tubing or pump



Touch Screen
touchscreen LCD with timer control



Ultra-Quiet
for discreet pumping sessions